

# Surgical Line



## OMNI-VAC

Collection system for organic waste liquids during surgical aspiration




# OMNI-VAC

The concept is very simple. Thanks to sheer gravity, extracted material is retained in the suction collection bag contained in the canister. Due to the created vacuum, liquids and particles of tissue from the treatment site are extracted and channelled into the collection bag. Blood, saliva and saline solution as well as particles of bone and surrounding tissue may cause blockages in your suction system. The suction system of your dental unit is in constant use throughout oral surgical and implantological interventions.

Its failure-free operation is imperative for uninterrupted and successful treatment. This is guaranteed by the new collection system for organic waste liquids. OMNI-VAC system retains the extracted material and, in doing so, effectively safeguards the pump filter of your dental unit against blockages. The extracted material remains in the collection bag that can be separated from the canister and disposed of.

The collection bag for organic waste liquids is disposable, while the Canister (1.5 Lt.) is autoclavable at 134°C. The Omnia Collection system for organic waste liquids during surgical aspiration can be connected directly to the dental unit.

## ASSEMBLY SCHEME

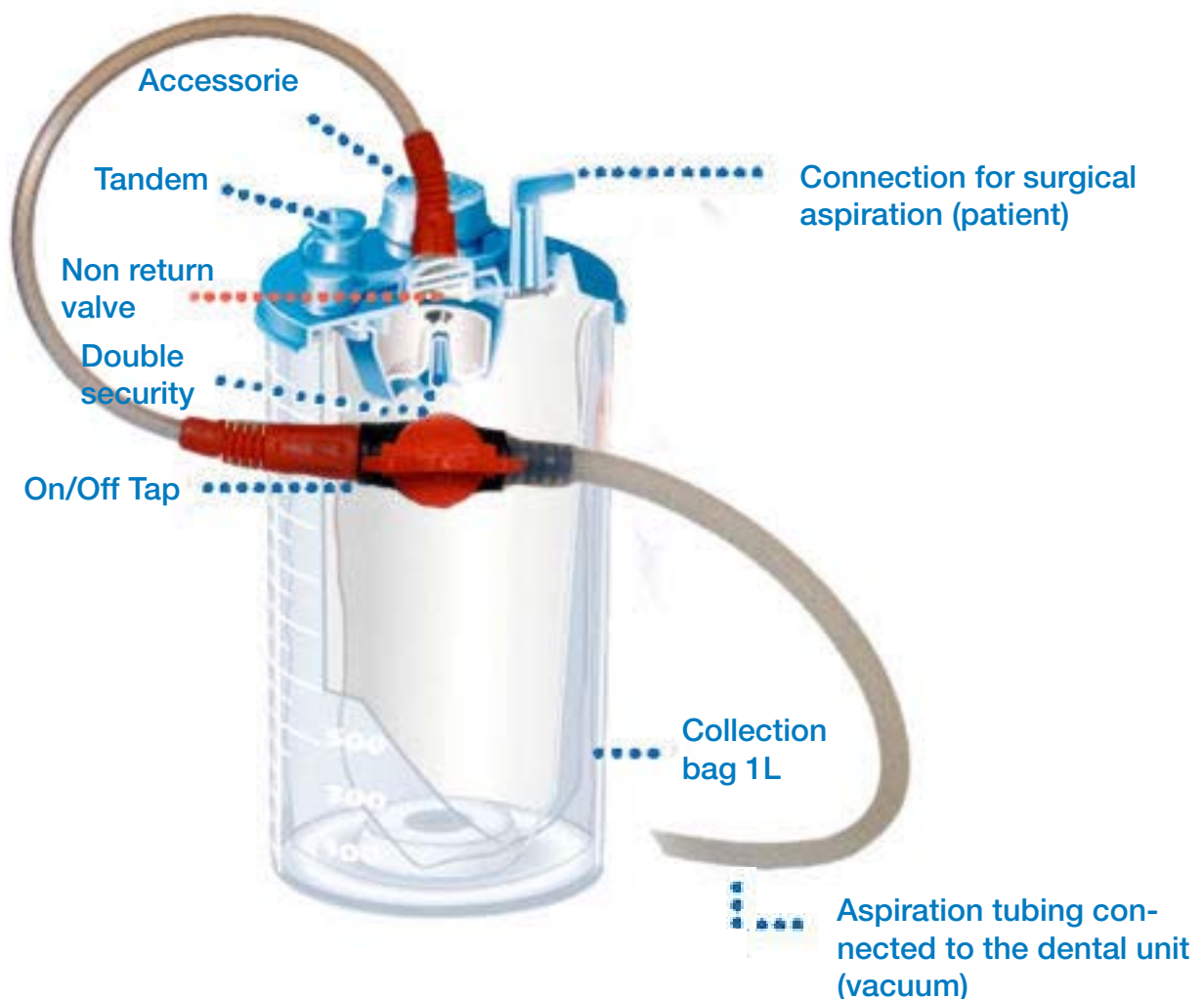

## SUPPORT OMNI-VAC

The OMNI-VAC support is made in stainless steel with wheeled base-ment. It allows the positioning of the OMNI-VAC aspiration system with the possibility of easily moving it in the surgery room.

It is possible to put 2 containers in parallel connection.

It is possible to use the tray to sup-port the Physiodispenser.



### 30.F2000

OMNI-VAC - Canister for suction collection bag for organic waste liquids. The box contains also a disposable collection waste bag and the connection tubing to the dental unit

### 30.F2001


Suction collection bag for organic waste liquids  
Polyethylene material - 10 pcs/box

### 30.E0027

Removable support for Canister for suction collection bag for organic waste liquids - Stainless steel material - 1 pc/box

### 30. E0014

Support with tray for physiodispenser with 10 kg capacity - 1 pc/box

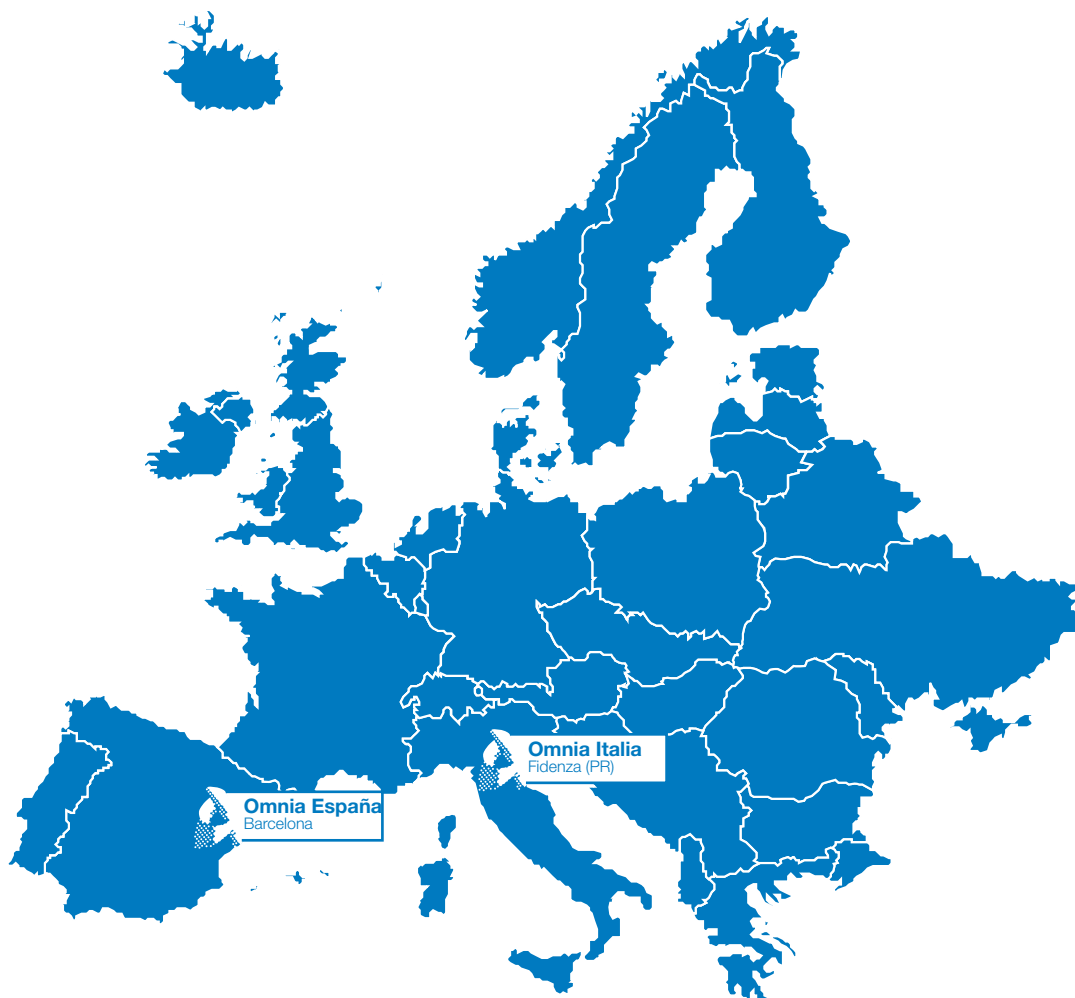

**OMNIA**<sup>®</sup>  
Disposable Medical Devices

**Omnia S.p.A.**

Via F. Delnevo, 190sx  
loc. S. Michele Campagna - 43036 Fidenza (PR) - Italy  
tel +39 0524 527453 - fax +39 0524 525230  
[www.omniasrl.com](http://www.omniasrl.com) / [info@omniasrl.com](mailto:info@omniasrl.com)

Partner **Omnia España S.L.**

Plaza Francesc Macià , 8 - 6ºD  
08029 Barcelona - España  
tel +34 93 405 31 19 - fax +34 93 439 31 42  
[www.omniaspain.com](http://www.omniaspain.com) / [pedidos@omnia-spain.com](mailto:pedidos@omnia-spain.com)

